



reducecept

Reducecept

Mobile manual

Reducecept

Blokhuisplein 40, 8911 LJ Leeuwarden, Nederland

E-mail: info@reducecept.com

Tel: +31 85 13 08 260

CoC: 74239678

VAT: NL859821432B01

www.reducecept.com

Reducecept is a part of RelieVR B.V.




Reducecept mobile app

The Reducecept mobile app allows you to play Reducecept without an Oculus Go headset. This manual explains how to install Reducecept on your smartphone and link your game to the subscription you've received from your practitioner.


Installing Reducecept on your phone

1. Open the iTunes Store (on iPhone) or the Google Play store (on Android phones).

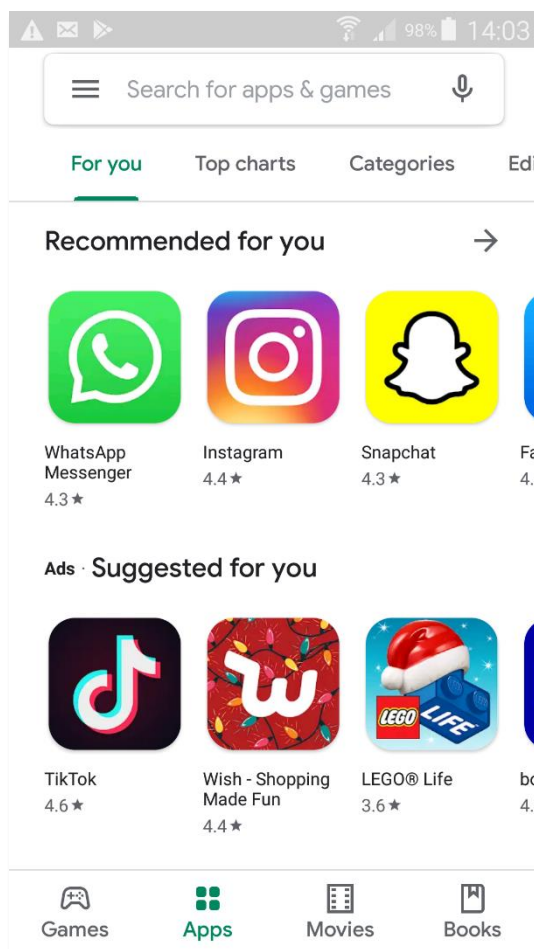




2. Search for 'reducept' in the search field. Choose the Reducept app in the search results.

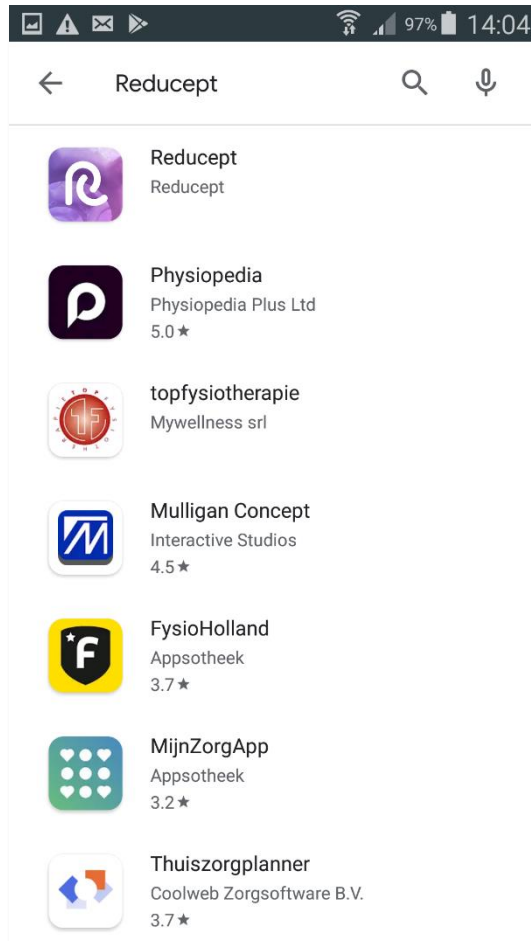




3. Press 'Install'.

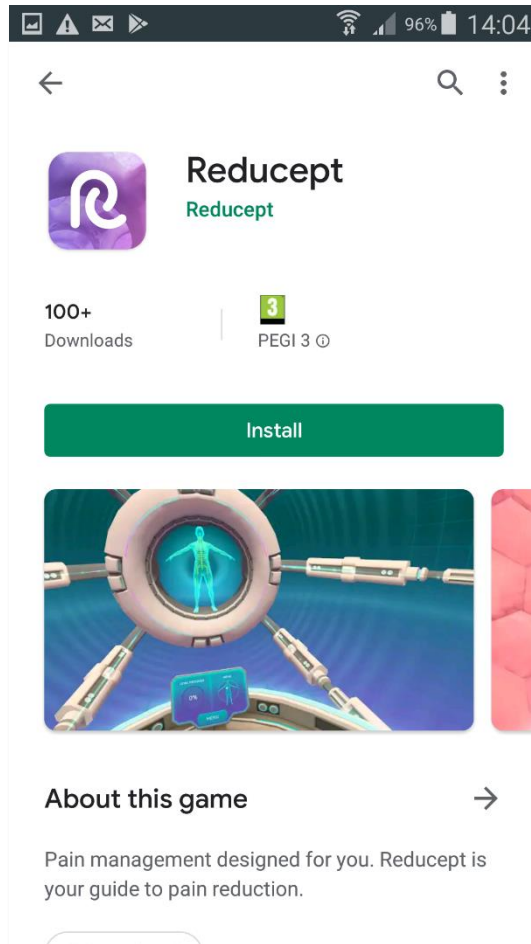



4. Press 'Open'. The Reducept app has also been added to the home screen of your phone.

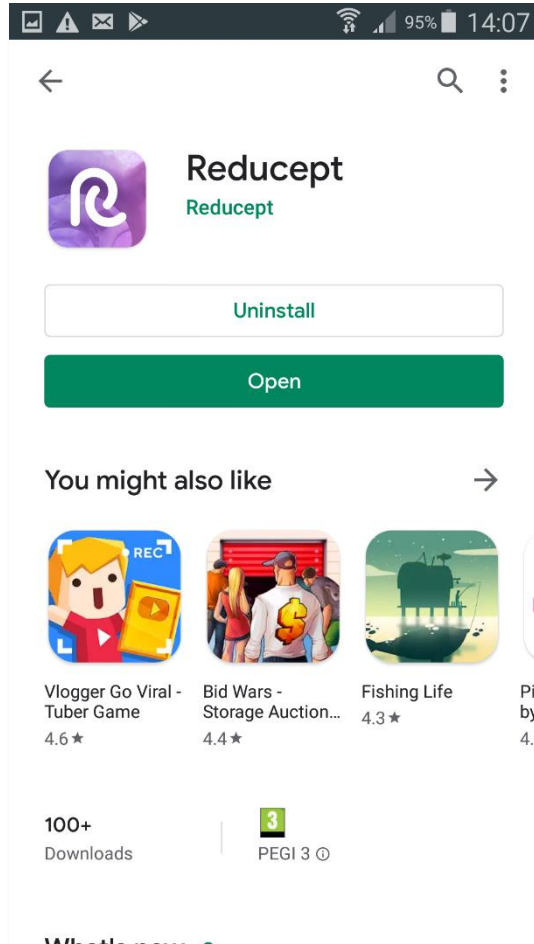




Logging into the Reducept app


1. Start the Reducept app (see the chapter 'Installing Reducept on our phone').

2. Press the arrow in the 'User' section of the main menu.

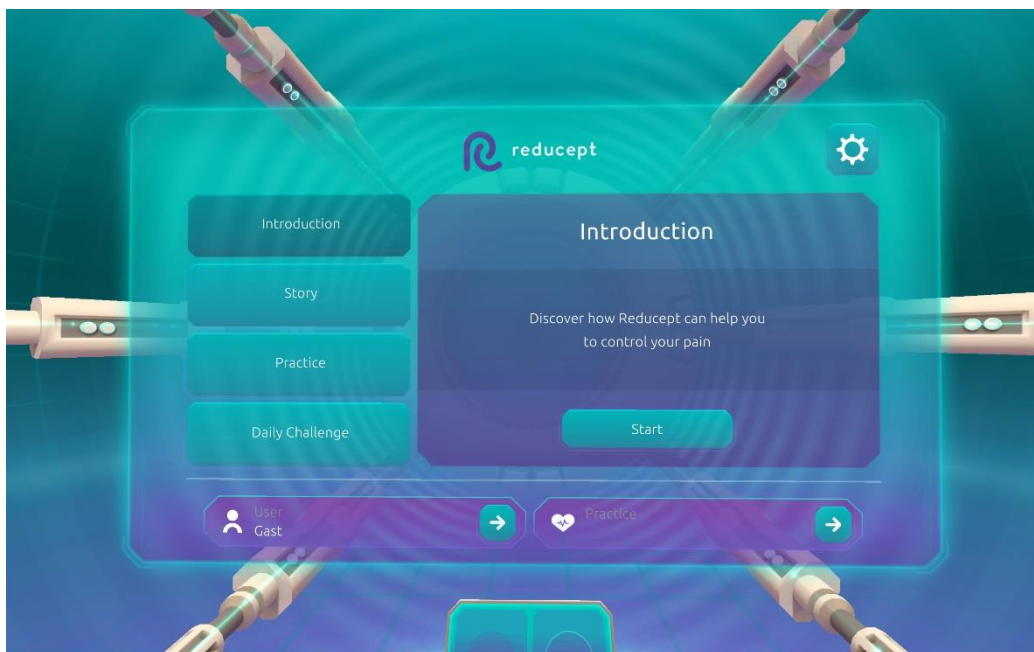




3. Enter the e-mail address and password of your Reducecept dashboard account and then aim your visor at the 'Log in' button.

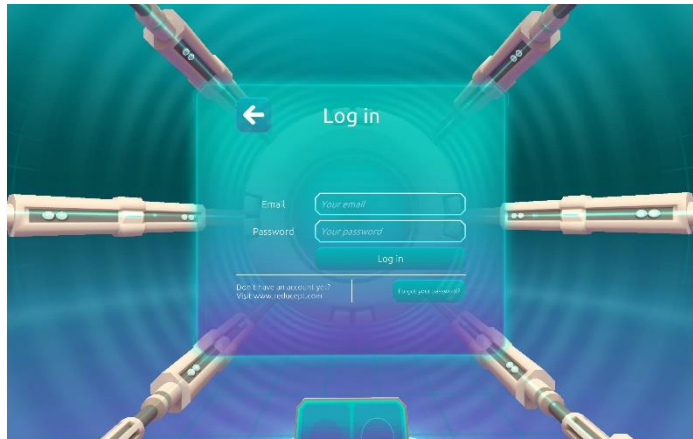

4. You are now logged in.

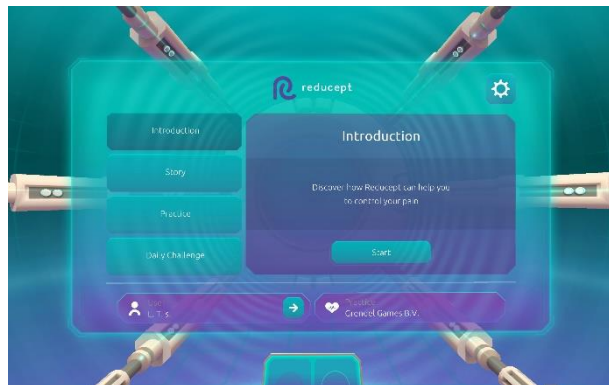




Playing the Reducept game in VR


1. Start the Reducept app (see the chapter 'Installing Reducept on your phone'). Make sure you are logged in (see the chapter 'Logging into the Reducept app').

2. Open the 'Options' screen by pressing on the gear button in the top right corner.

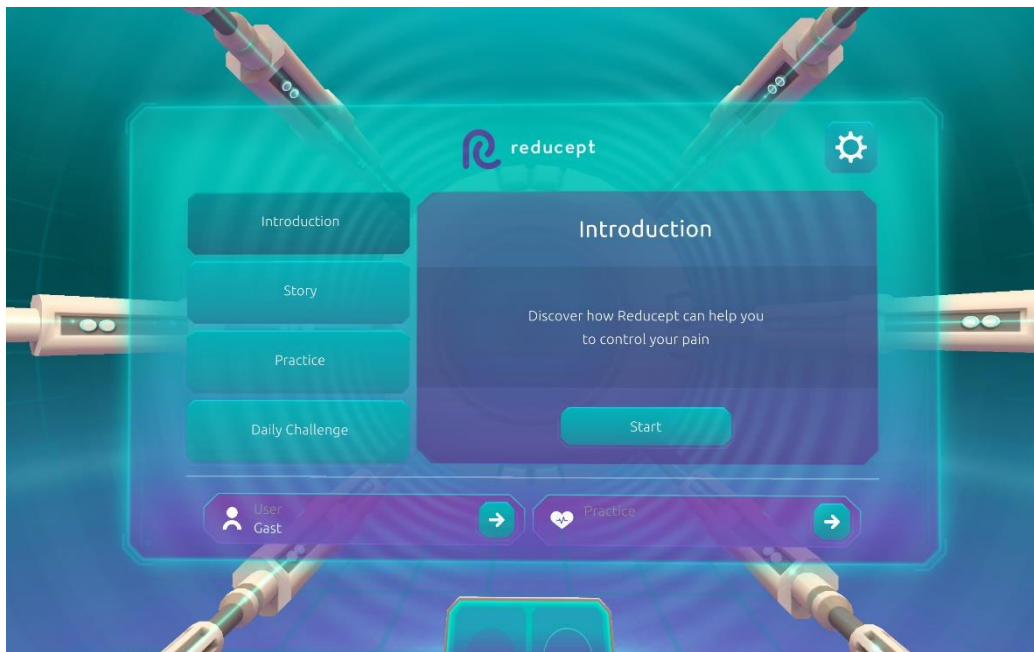



3. In the 'Options' screen, press the 'Enable VR' button.

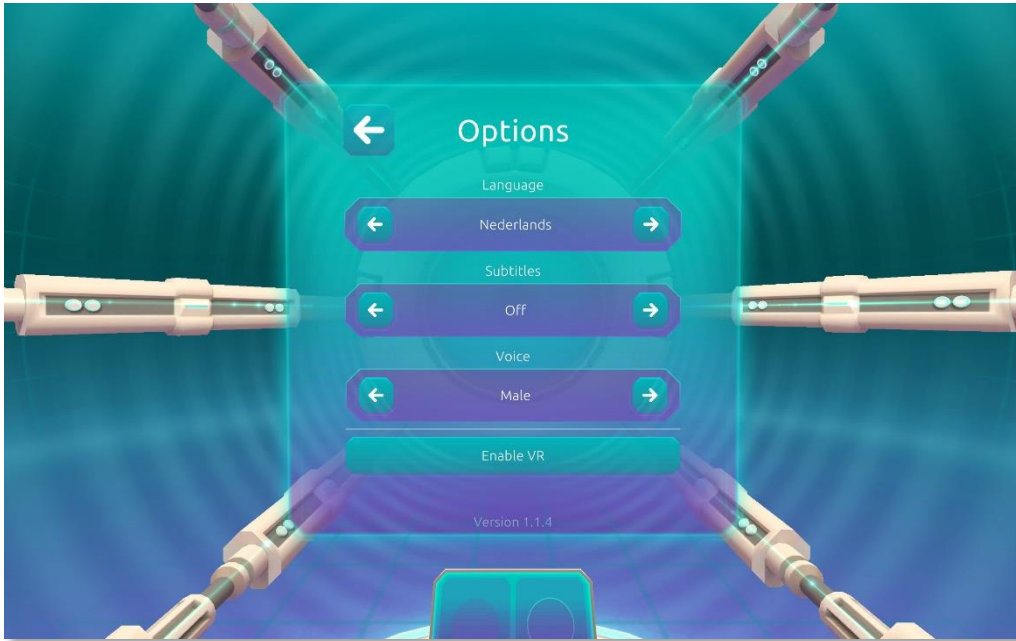

4. Place your phone in a suitable VR headset and place the headset on your head.

